

Start a New Revolution

Needless catheter securement device,

with easy viewing, protection and cleaning.

REVOLUTION™

REVOLUTION™

CATHETER SECUREMENT DEVICE

A revolutionary new device that saves time, provides convenience and enhances compliance with safety regulations by securing catheters with a needleless procedure.




STEP ONE
PEEL off the paper backing.



STEP TWO
CENTER and APPLY Revolution to skin that is clean, dry, and free of any solvents. Pass the catheter through the center of the Revolution and apply to the skin.



STEP THREE
PULL FINGER RINGS to RELEASE the first set of strings until they are snugly deployed around the catheter. Do not pull hard, this can kink or cut the catheter.

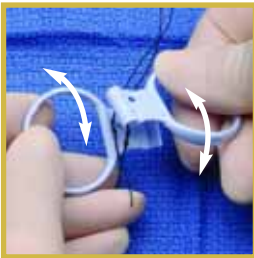

STEP FOUR
TWIST back and forth to remove finger rings.



STEP FIVE
ROTATE THE BLUE RING clockwise to deploy the next set of strings. Stop rotation when strings wrap snugly around the catheter. DO NOT OVER TIGHTEN. Place your thumb at the catheter tie-down post prior to rotating the blue ring. This allows for easier turning and removes pressure from bandage.



STEP SIX
TIE off bottom strings around catheter using a surgeon's knot. DO NOT OVER TIGHTEN. This may cause damage to the catheter or restrict flow.

ONE DEVICE FOR VARIOUS SIZE CATHETERS

For drainage catheter sizes 6F – 24E

CENTRAL ADHESIVE HYDROCOLLOID
Skin friendly, absorbent around puncture site, conforms well to body anatomy.

NON-WOVEN MATERIAL
with ribbed coated adhesive secures to skin for up to 7 days with comfortable adhesive.



COVER LID
Provided to cover the puncture site while showering for added protection.

ORDERING INFORMATION

CFG624

Box qty. 25 units
Supplied sterile, single pouch.
Intended for one-time use.

LATEX FREE

